

PrecisionBioLogic

CRYOcheck™ Frozen Product Storage and Handling

Get the most from your **CRYOcheck** frozen products by using our simple storage and handling guidelines in conjunction with your lab's standard operating procedures.


Product Packing & Delivery

All Precision BioLogic frozen products come in compact, color-coded packaging for easy location in your freezers. To ensure product integrity during shipment, we carefully pack and monitor orders until they arrive safely at your site.




Unpacking your Order

Your order is shipped in an insulated shipping carton with dry ice. Please follow your workplace procedures for handling and disposing of dry ice.

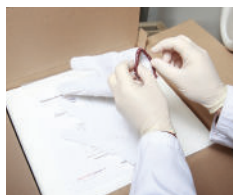

Your shipment of **CRYOcheck** frozen products includes:

- gloves to keep your hands warm and safe while handling dry ice and frozen products
- applicable lot-specific information sheets for new lot numbers. These documents are also available on our portal (<https://my.precisionbiologic.com>).
- a temperature tag that will activate if the temperature in the shipping carton rises above -18°C / 0°F

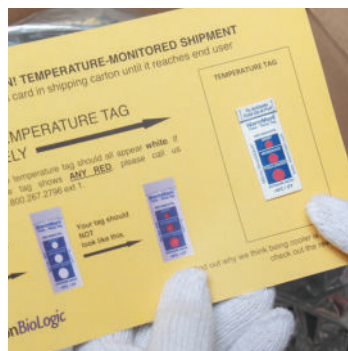

As soon as you receive your shipment, please check the temperature tag.

- **If the circles on your tag show any red, please call us immediately at 1.800.267.2796 x 1 for further instructions.**
- **If the circles are all white, put your frozen products away immediately in an appropriate freezer.**

Note: Some products are shipped in sealed mylar bags to protect them from dry ice. Once a mylar bag has been opened, product must be immediately removed from dry ice and put in an appropriate freezer.


CRYOcheck Frozen Product Storage and Handling

Storing your CRYOcheck Frozen Products



To ensure product integrity and performance, always store your CRYOcheck frozen products at the temperatures recommended by Precision BioLogic.

Storage temperatures vary by product and are noted on product boxes and Instructions For Use (IFU).

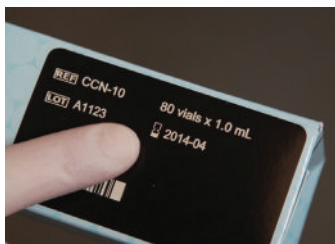

When stored at the recommended temperature, CRYOcheck frozen products are stable to the end of the month indicated on the product packaging.




Do not store CRYOcheck frozen products in a frost-free freezer. These freezers are engineered to cycle through frequent temperature fluctuations, which will negatively affect product performance.



For your convenience, we provide you with gloves to keep your hands warm while handling frozen products and an ice scraper to remove frosty build up in your freezers.

Thawing your CRYOcheck Frozen Products

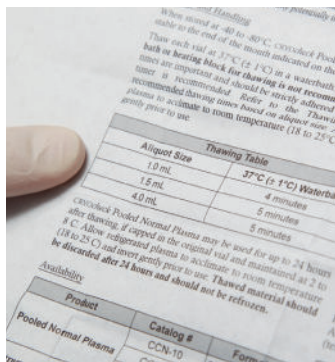

1 Prepare a waterbath, ensuring the water temperature is 37 °C (±1 °C). **Note: Do not use a dry bath or heating block for thawing, it may adversely impact product performance.**

2 Refer to the IFU for the appropriate thawing time based on aliquot size.



3 Thaw each vial at 37 °C (±1 °C) for the recommended time.

Note: Thawing times are important and should be strictly adhered to. The use of a timer is recommended.

For your convenience, Precision BioLogic provides foam “floaties” to keep vials upright in the waterbath, along with a timer.

4 Once thawed, refer to the IFU for further preparation instructions.

Storing Thawed CRYOcheck Products

Refer to the IFU for information about storing a thawed product.

Any questions? We’re happy to help, call us for more information.

PrecisionBioLogic

www.precisionbiologic.com

T 1.800.267.2796 / +1 902.468.6422

F 1.800.267.0796 / +1 902.468.6421